maniquick®

Quality Tradition & Innovation

NEW
MORE
EFFECTIVE!

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TABLE OF CONTENTS

- Introduction
- Safety Instructions
- Special Features
- Description of the Appliance & the Accessories included
- Battery Installation
- Cleaning & Maintenance
- Specifications
- Operation
- Warranty

INTRODUCTION

Head Lice are wingless insects which, when full grown, are about the size of a sesame seed. Head Lice cannot survive the cold so they cling to the hair close to the scalp for warmth. These parasites have been around for thousands of years and are harmless, although they can cause severe scalp itching. Fortunately, they can now be completely eliminated with only a ten minute treatment.

The **Maniquick Electric Lice Comb** is designed to easily eradicate Head Lice and prevent spreading to other family members. It is effective on Head Lice and on Nits, once they have reached sufficient size. The parasite must be of sufficient size to come in simultaneous contact with two teeth of the Comb. This is easier with the Louse which averages 2-3 mm in size, but is more difficult with the Nit, which is smaller, averaging only 1mm. For this reason the **Maniquick Electric Lice Comb** should be used for several consecutive days, increasing the chance to eradicate the Nits as they grow in size. It is recommended that the **Maniquick Electric Lice Comb** be used one or twice a week for a period in order to prevent re-infestation.

SAFETY INSTRUCTIONS

It is recommended that you read the User Guide very carefully and comply with the instructions in order to prevent any accidents or damage to the unit.

- Use the Appliance only as intended and in accordance with the instructions supplied in the guide
- This Electric Device is not a toy and should be kept out of the reach of children
- This **Maniquick Electric Lice Comb** should not be used on children under the age of 3.
- Children should be supervised by an adult when using this device.
- If you have a Pace Maker, Artificial Limbs or any electric implant, consult a physician before using.
- Do not operate this unit within 1 meter of a mobile telephone, microwave oven or television set.
- It is not recommended for use by anyone suffering from Epilepsy or Heart Disease.
- Do not use the **Maniquick Electric Lice Comb** on wet or damp hair. Be sure that the hair is unbraided and flowing loose and remove all hair pins or ornaments.
- Avoid contact with cracked or dry skin and keep away from ears, eyes and mouth.
- Avoid placing the unit in direct sunlight or placing it on a hot surface.
- Keep the unit away from liquid or any form of moisture.
- Do not use the unit if the housing or comb are damaged, if the unit is not functioning properly or if it has been immersed in liquid.
- Prevent accidents and damage to the unit by keeping it away from clothing, bristles, wires, strings, shoe laces, threads, etc.
- Always switch the unit off when not in use.
- Remove the battery if the unit is not to be used for an extended period.

SPECIAL FEATURES

Variable Voltage Range

The **Maniquick Electric Lice Comb** efficiently eliminates Head Lice and Nits with its unique variable voltage system that has a range of 130 to 180 volts. While effective in killing these parasites, it is completely harmless to humans.

Audible Signals Indicating Functions

Comb activity is indicated by audible sounds that indicate different functions. A sound every three seconds indicates that the unit is functioning. A short buzzing sound every ½ second indicates contact with the parasite. The LED setting at the On/Off switch blinks to indicate different functions and to indicate that the battery is low.

Automatic Shut Off Feature

The unit will automatically switch off if not in use for 5 minutes, thus extending battery life.

Low Battery Indicator


If the battery is low and cannot generate the proper voltage, the LED indicator will make 2 short and one long blink, the buzzer will sound for 30 seconds and the unit will automatically switch off.

Unique Design

The unit is specially designed to contain the accessories and for portability and easy storage.


DESCRIPTION OF APPLIANCE & ACCESSORIES

1. On/Off Switch
2. Protective Cover
3. Electric comb
4. Battery Compartment Lid
5. Cleaning Brush
6. Battery Compartment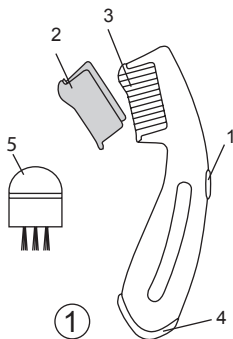

BATTERY INSTALLATION

1. Open the Battery Compartment Lid by lifting at point "4". The lid is hinged to prevent loss.
2. Insert two AAA batteries, making sure to observe the correct polarity (+/-) as indicated on the batteries and on the Battery Compartment. The wrong polarity can cause damage to the unit and will void the warranty.
3. Close the Battery Compartment Lid and listen for the buzz to indicate that the batteries have been changed correctly.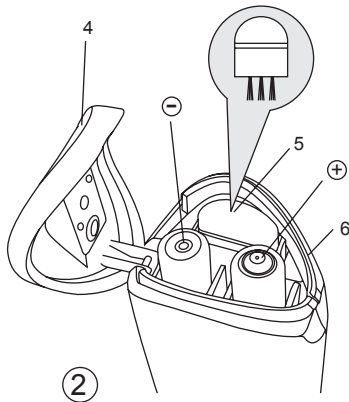

CLEANING & MAINTENANCE



1. Remove batteries before cleaning the appliance
2. Do not use any strong detergents or other liquids to clean the unit
3. Use a soft paper or cloth to clean the housing
4. Use the Accessory Brush provided to clean the comb
5. Make sure that no water gets into the unit while cleaning and do not immerse in water.
6. Store in a clean, cool and dry area that is well ventilated.
7. Direct exposure to sunlight could damage the unit.

SPECIFICATIONS

Model name	MQ003
Power supply	AAA battery x 2
Standby Current	$\geq 30\mu\text{A}$
Full Current	$\geq 30\text{mA}$
Export Voltage	30~180 V
Consumption	$\geq 35\text{mA}$ during use
Battery life	8 hours in continuous use
Auto switch off	Yes
Dimension	177 x 34 x 39 mm
Weight	200g
Certificate	CE, RoHS

OPERATION

1. Undo braids and remove any hair ornaments.
2. Press the On/Off button "1" for two seconds to turn the unit on. The LED will light and the buzzer will sound for three seconds, indicating that the unit is ready for use.
3. Use the **Maniquick Electric Lice Comb** as you would a normal comb, making sure that hair is completely dry.
4. When the unit is operating the LED light will be on and the buzzer will sound every 3 seconds.
5. The Buzzer will sound and the LED will flash when the comb comes in contact with a Head Louse or Nit.
6. After use, be sure to switch off the unit, clean the comb with the brush provided and replace the Protective Cap "2" before storing the unit. Once the unit is switched off you may hear a buzzing sound as the unit is discharging. The sound will stop once the unit has discharged completely.

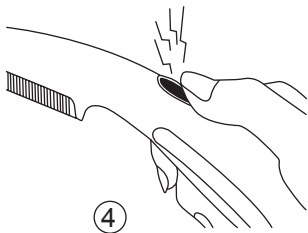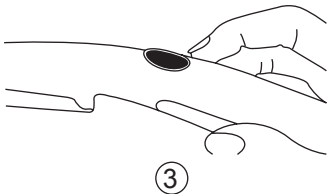CONDITIONS FOR SERVICE UNDER GUARANTEE

1. SANICO Srl guarantees the absence of faults and mechanical defects and the completeness of the product when it is sold.
2. Repairs can be done only at Service Centres authorized by SANICO Srl or by the Authorized Distributor in the country where you have purchased the appliance.
3. The duration of the period of guarantee on the materials and on manufacturing and operating defects is 2 years with effect from the date of sale, subject to all the conditions for the Purchaser shown in the Instructions for use. The obligations of guarantee do not cover expendable materials: electrode cups, elastic bands, power supply cables etc..
4. Complaints relative to the product must be presented directly to the authorized service centre. The service centre, on the basis of an agreement with the Vendor, is competent to establish the cause and nature of the fault. The product will be replaced only if it cannot be repaired.
5. The owner will provide to deliver the product to the after-sales service center, as per the delivery instructions that can be requested by e-mail: info@maniquick.com.
6. The time necessary for the repair under guarantee will be subject to the degree of seriousness of the fault.
7. The responsibility of SANICO Srl under these conditions of guarantee will be limited to the amount paid by the Purchaser for the product in question.

The guarantee on the product will not be applied in the following cases

- certificate of guarantee filled in incorrectly or falsified; serial number (if applicable for the specific product) damaged or removed;
- absence of commercial and financial documents (cash receipt, delivery note) proving the purchase;
- opening or repair of the product by the Purchaser or by an unauthorized Service Centre;
- defects following failure by the Purchaser to observe the rules for use, storage and transport of the product shown in the relative technical and regulations documentation, in which case the regulations of the aforementioned documentation which have been infringed must be specified;

- penetration of foreign bodies or liquids inside the product;
- presence on the product of internal or external mechanical defects (cracks, signs of a fall, chipped parts, etc.) consequent to the incorrect use, installation or transport of the product;
- components or accessories subject to rapid wear and tear (electrode cups, elastic bands);
- defects caused by external agents (dirt visible externally and internally, traces of corrosion);
- defects due to causes independent of the manufacturer, such as power supply voltage step changes, natural phenomena or disasters, fires, actions by animals, insects etc.
- complaints for defects found by the Purchaser that are made after the expiry of the guarantee period or any other period established by the law for reporting any defects;
- use of the product by the Purchaser not for personal (domestic) use but for commercial and profit-making purposes.

Conditions for the return of a defective product

Defective products will be accepted only if clean and complete, in the original packaging, together with the certificate of guarantee correctly filled in and the commercial and financial documents (cash receipt of the store, delivery note) that prove the purchase. Please attach a description of the problem found to the appliance.

After Sales Service Center, please contact:

e-mail: info@maniquick.com

www.maniquick.com

Before sending the appliance, please contact us by phone or e-mail to agree on how the product is to be returned

Send this coupon in case of repairs:

Product: MQ003 Lice Comb

Serial Number:

if available

Date of purchase:

--	--	--	--	--	--	--	--	--	--

Dealer's Stamp:

--

Buyer's Full Name:

Street/Square:

N°:

City and State:

Postal Code:

Country:

Phone Number:

E-mail:

@

Problem description:

Signature:

Date:

--	--	--	--	--	--	--	--	--	--

WARRANTY IS VALID ONLY IF ACCOMPANIED BY INVOICE/TICKET.

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SECURITY OF ALKALINE BATTERIES INTO PRODUCTS

1. Fire - Never delete the batteries throwing them in a fire. they could break and release the substances contained inside them.
2. If ingested or inserted, seek immediate medical assistance.
3. Do not store or leave the batteries in places accessible by children.
4. Remove old or weak batteries from appliances.
5. Do not mix old and new batteries in the same apparatus. There may be losses. The replacement of all batteries in a device must be done simultaneously.
6. Never mix different types of batteries in a single device. There may be losses.
7. Install carefully the batteries in the correct polarity (+ and -) according to the instructions provided by the manufacturer of the product
8. Do not leave the Products turn on with batteries when they are depleted. There may be losses.
9. Remove the batteries from any device that must remain unused for long periods of time
10. Avoid contact of the batteries with metal objects or other batteries because they may experience losses or they could break.
11. Do not recharge batteries because there may be some losses or they could break.
12. Not groped to disassemble the batteries
13. Store batteries in a cool and dry. Avoid sudden changes in temperature. Store the batteries in their original packaging before use.
14. RESPECT THE ENVIRONMENT - DISPOSE OF EXHAUSTED BATTERIES IN THE CONTAINERS

The battery and the possible Electronic Equipment -AEE- or anything that contains it must not be disposed as urban waste, but in compliance with Directive 2002/96/EC, 2006/66/EC and of National Laws transposition for the disposal of the product end of life, and must be subjected to a separate collection to avoid damage to the environment. Failure to comply with the above is sanctioned by Law.

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DISPOSAL OF YOUR OLD APPLIANCE

When this crossed-out wheeled bin symbol is attached to a product, it means the product is covered by the European Directive 2002/96/EC.

All electrical and electronic products should be disposed off separately from the municipal waste stream via designated collection facilities appointed by the government or the local authorities.

The correct disposal of your old appliance will help prevent potential negative consequences for the environment and human health.

For more detailed information about disposal of your old appliance, please contact your city office, waste disposal service or the shop where you purchased the product.

IMPORTATO DA:

SANICO Srl

Via G.Ferraris, 37/41

20090 Cusago (MI) ITALY

Tel (+39) 02.90.39.00.38 - Fax (+39) 02.90.39.02.79

info@sanicare.it - www.sanicare.it