maniquick®
WELLNESS & BEAUTY

BILANCIA MULTIFUNZIONALE
MULTIFUNKTIONSWAAGE
MULTIFUNCTIONAL SCALE
Многофункциональные весы

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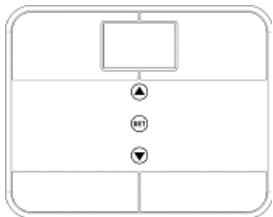
WARRANTY IS VALID ONLY IF ACCOMPANIED BY INVOICE/TICKET.

Thank you for choosing our electronic body fat scale.
To ensure proper operation and to maintain the life and accuracy of the scale, please read this user's guide carefully.

Key features:

- This product adopts advanced BIA (bioelectrical impedance Analysis) technology. BIA measures body fat, hydration and muscle percentage, bone weight, KCAL parameter by sending a safe, low-level electrical signal through the body. This signal is analyzed by a computer to determine the body fat, hydration and muscle percentage, bone weight, KCAL parameter. It is more precise than traditional calculations.
- Use high precision sensor to obtain more exact weight.
- Use high quality tempered glass and elegant support feet for sleek modern style.
- Personal data can be stored in the memory of the unit for up to 10 people. It is very convenient to use.

Button operation

SET key: power on, menu select and confirm

▲Key: Up

▼Key: Down

Operation

- Open the battery cover on scale base, and install the batteries.

- Put the scale on hard flat floor. Make sure it level.
- Power On methods.
 - 1) Auto Power On: weighing when step on.
 - 2) Manual Power On: touch "SET" over 2 seconds.
- After power on the scale and showing "0.0" on the display, press "Up" or "down" key to change the display UNIT, the display units will switch kg, st:lb and lb, then to use.

Note:

For the first time using, long time no using or moving the scale to other place, you must to restart the scale before using. If not, the result will be not exact.

How to measure your body weight

Weight only:

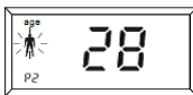
1. Stand on the scale erectly and maintain your balance. Display will indicate your weight.
2. After weighing, the scale will be auto off if no further using.

1. Setting and storing personal data in memory

- 1.1 Press "SET" key. Use "Up" or "Down" key to select your personal key form P0-P9. Once you reach the personal key you wish to use, Press "SET" to confirm.

1.2 Press “▲”or “▼” to choose Male or female, then press “SET” to confirm

1.3 Press “▲”or “▼” to choose age (10-80), press “SET” to confirm.

1.4 Press “▲”or “▼” to choose height (100-220), “press “SET” to confirm.

1.5 After finishing the step 1.4, the scale will display “0.0”, it is ok to measure weight, body fat, hydration and muscle percentage, bone weight, KCAL parameter.

2 Taking weight, body fat, hydration and muscle percentage, bone weight, KCAL parameter readings.

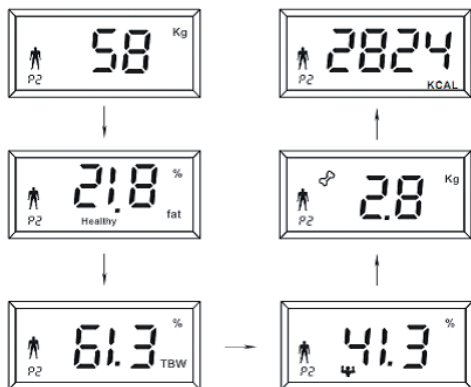
2.1 When you use the scale, use your personal data code number. Press “SET” to power on, the display show “0.0”, and press “▲”or “▼” to select your personal data number. Then press “SET” key to confirm. The scale will indicate “0.0”.

2.2 Stand by bare foot on scale, maintain contact with electrodes.

2.3 Your weight will be display first. Continue to stand erectly, then your body fat, hydration and muscle percentage, bone weight, KCAL parameter will appear on the display showing three times after flash "0000"

Parameter reference

2.4 Body fat percentage normal ranges:

Body fat ranges for standard adults

Age	Female			
	Body Fat Percentage			
	Underfat	Healthy	Overfat	Obese
12-20	<16%	18-28%	28-33%	>33%
21-42	<20%	20-30%	30-35%	>35%
43-65	<21%	21-31%	31-36%	>36%
66-100	<22%	22-32%	32-37%	>37%

Age	Male			
	Body Fat Percentage			
	Underfat	Healthy	Overfat	Obese
12-20	<15%	15-21%	21-26%	>26%
21-42	<17%	17-23%	23-28%	>28%
43-65	<18%	18-24%	24-29%	>29%
66-100	<19%	19-25%	25-30%	>30%

2.5 Body hydration rate normal ranges:

The comparison sheet of hydration measured and hydration standard level

Age	Male	Female	Corresponding Denotation	
<=30	66,0-60,5%	66,0-59,9%	Optimal Hydration	Weight scope of hydration: 37.8-66.0%
	60,4-57,1%	59,8-56,4%	Slight Dehydration	
	57,0-53,6%	56,3-53,0%	Moderate Dehydration	
	53,5%-50,2%	52,9-48,6%	Dehydration	
	50,1-37,8%	48,5-37,8%	Severe Dehydration	
>30	66,0-59,1%	66,0-55,0%	Optimal Hydration	Hydration increment 0.1%
	59,0-55,7%	54,9-51,6%	Slight Dehydration	
	59,6-52,3%	51,5-48,1%	Moderate Dehydration	
	52,2-48,8%	48,0-44,7%	Dehydration	
	48,7-37,8%	44,6-37,8%	Severe Dehydration	

Muscle rate normal ranges for reference

Male			
Age	low	normal	high
20-29	<42%	42-54%	>54%
30-39	<41%	41-52%	>52%
40-49	<40%	40-50%	>50%
50-59	<39%	39-48%	>48%
60-69	<38%	38-47%	>47%
da 70	<37%	37-46%	>46%

Female			
Age	low	normal	high
20-29	<34%	34-39%	>39%
30-39	<33%	33-38%	>38%
40-49	<31%	31-36%	>36%
50-59	<29%	29-34%	>34%
60-69	<28%	28-33%	>33%
da 70	<27%	27-32%	>32%

Body bone normal ranges for reference

Female			
Body weight	Under 45 kg	45-60 kg	Over 60 kg
Bone weight	>1,8 kg	>2,2 kg	>2,5 kg

Male			
Body weight	Under 60 kg	60-75 kg	Over 75 kg
Bone weight	>2,5 kg	>2,9 kg	>3,2 kg

KCAL-Parameter:

KCAL parameter display scope: 500-5000

The KCAL parameter indicates that a person need how much Kcal everyday, which also lie on different sport time.

Because of differences in geographic areas and locations, the body fat, hydration, muscle, bone Kcal, standards can be vary. This sheet is for reference only.

Please remember your personal key. You do not need to program personal key the next time you use scale. You can use the personal key you have already stored.

Caution

1. Personals with pacemakers should not use this product.
2. Personals with metal implants will be unable to obtain accurate readings.
3. To ensure accuracy, we suggest using scale at the same time every night (19:00-21:00).
4. "Lo" on scale display indicate that the battery is out of power. You should install a new one.

5. The "Err" on scale display indicates overload scale "Err2" display indicates fat percentage measurement error.
6. If body fat percentage is less than 5%, indicator displays "Err2" If more than 50% indicator displays "Err2" If impedance measuring makes an error, indicator will display "Err2".

INTRODUCTION OF USING AND INSTALLING THE BATTERIES

- Use 3X1.5V AAA batteries (3 PCS).
- When change the batteries, please open the battery cover on the back, put 3 PCS 1.5V AAA batteries in the battery case (make the positive poles towards), then close the battery cover tight.

Note

1. You should place scale on clear, flat surface before use.
2. Do not place scale on wet, severely hot or extremely cold surfaces.
3. Do not jump or stomp on scale. Do not attempt to disassemble the scale.
4. To avoid falling off scale, do not stand on the scale to avoid glass and electrode wear out .
5. Do not wear shoes when standing on the scale to avoid glass and electrode wear out.
6. Never submerge the unit in water. Use alcohol to clean the electrodes and glass cleaner to keep them shiny, applied to a cloth first; avoid soaps.
7. For home use only. Not for professional use. Body fat percentage and hydration percentage is reference only. You should consult a doctor when you undertake any diet or exercise program.
8. Check battery if scale malfunction. If necessary, install a new battery. For other problems, please contact your local franchiser or contact our company for repair.

CONDITIONS FOR SERVICE UNDER GUARANTEE

1. SANICO Srl guarantees the absence of faults and mechanical defects and the completeness of the product when it is sold.
2. Repairs can be done only at Service Centres authorized by SANICO Srl or by the Authorized Distributor in the country where you have purchased the appliance.
3. The duration of the period of guarantee on the materials and on manufacturing and operating defects is 2 years with effect from the date of sale, subject to all the conditions for the Purchaser shown in the Instructions for use. The obligations of guarantee do not cover expendable materials: electrode cups, elastic bands, power supply cables etc..
4. Complaints relative to the product must be presented directly to the authorized service centre. The service centre, on the basis of an agreement with the Vendor, is competent to establish the cause and nature of the fault. The product will be replaced only if it cannot be repaired.
5. The owner will provide to deliver the product to the after-sales service center, as per the delivery instructions that can be requested by e-mail: info@maniquick.com.
6. The time necessary for the repair under guarantee will be subject to the degree of seriousness of the fault.
7. The responsibility of SANICO Srl under these conditions of guarantee will be limited to the amount paid by the Purchaser for the product in question.

The guarantee on the product will not be applied in the following cases:

- certificate of guarantee filled in incorrectly or falsified; serial number (if applicable for the specific product) damaged or removed;
- absence of commercial and financial documents (cash receipt, delivery note) proving the purchase;
- opening or repair of the product by the Purchaser or by an unauthorized Service Centre;
- defects following failure by the Purchaser to observe the rules for use, storage and transport of the product shown in the relative technical and regulations documentation, in which case the regulations of the aforementioned documentation which have been infringed must be specified;

- penetration of foreign bodies or liquids inside the product;
- presence on the product of internal or external mechanical defects (cracks, signs of a fall, chipped parts, etc.) consequent to the incorrect use, installation or transport of the product;
- components or accessories subject to rapid wear and tear (electrode cups, elastic bands);
- defects caused by external agents (dirt visible externally and internally, traces of corrosion);
- defects due to causes independent of the manufacturer, such as power supply voltage step changes, natural phenomena or disasters, fires, actions by animals, insects etc.
- complaints for defects found by the Purchaser that are made after the expiry of the guarantee period or any other period established by the law for reporting any defects;
- use of the product by the Purchaser not for personal (domestic) use but for commercial and profit-making purposes.

Conditions for the return of a defective product

Defective products will be accepted only if clean and complete, in the original packaging, together with the certificate of guarantee correctly filled in and the commercial and financial documents (cash receipt of the store, delivery note) that prove the purchase. Please attach a description of the problem found to the appliance.

After Sales Service Center, please contact:

e-mail: info@maniquick.com
www.maniquick.com

Before sending the appliance, please contact us by phone or e-mail to agree on how the product is to be returned

Send this coupon in case of repairs:

Product: MQ919

Serial Number:

if available

Date of purchase:

			/				/				
--	--	--	---	--	--	--	---	--	--	--	--

Dealer's Stamp:

--

Buyer's Full Name:

Street/Square:

N°:

City and State:

Postal Code:

Country:

Phone Number:

E-mail:

@

Problem description:

Signature:

Date:

			/				/				
--	--	--	---	--	--	--	---	--	--	--	--

WARRANTY IS VALID ONLY IF ACCOMPANIED BY INVOICE/TICKET.

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DISPOSAL OF YOUR OLD APPLIANCE

When this crossed-out wheeled bin symbol is attached to a product, it means the product is covered by the European Directive 2002/96/EC.

All electrical and electronic products should be disposed off separately from the municipal waste stream via designated collection facilities appointed by the government or the local authorities.

The correct disposal of your old appliance will help prevent potential negative consequences for the environment and human health. For more detailed information about disposal of your old appliance, please contact your city office, waste disposal service or the shop where you purchased the product.

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SECURITY OF ALKALINE BATTERIES INTO PRODUCTS

1. Fire - Never delete the batteries throwing them in a fire. they could break and release the substances contained inside them.
2. If ingested or inserted, seek immediate medical assistance.
3. Do not store or leave the batteries in places accessible by children.
4. Remove old or weak batteries from appliances.
5. Do not mix old and new batteries in the same apparatus. There may be losses. The replacement of all batteries in a device must be done simultaneously.
6. Never mix different types of batteries in a single device. There may be losses.
7. Install carefully the batteries in the correct polarity (+ and -) according to the instructions provided by the manufacturer of the product
8. Do not leave the Products turn on with batteries when they are depleted. There may be losses.
9. Remove the batteries from any device that must remain unused for long periods of time
10. Avoid contact of the batteries with metal objects or other batteries because they may experience losses or they could break.
11. Do not recharge batteries because there may be some losses or they could break.
12. Not groped to disassemble the batteries
13. Store batteries in a cool and dry. Avoid sudden changes in temperature. Store the batteries in their original packaging before use.
14. RESPECT THE ENVIRONMENT - DISPOSE OF EXHAUSTED BATTERIES IN THE CONTAINERS

The battery and the possible Electronic Equipment -AEE- or anything that contains it must not be disposed as urban waste, but in compliance with Directive 2002/96/EC, 2006/66/EC and of National Laws transposition for the disposal of the product end of life, and must be subjected to a separate collection to avoid damage to the environment. Failure to comply with the above is sanctioned by Law.

IMPORTATO DA:
SANICO Srl
Via G.Ferraris, 37/41
20090 Cusago (MI) ITALY
Tel (+39) 02.90.39.00.38 - Fax (+39) 02.90.39.02.79
info@sanicare.it - www.sanicare.it